



STERIKING® AC6 Strips

The STERIKING® indicator strips are manufactured in accordance with Good Manufacturing Practice Regulations promulgated by Food and Drug Administration (FDA) in USA for use as a monitoring indicator of the sterilization process carried out for the items requiring sterilization, such as surgical instruments and dressings in hospitals.

The STERIKING® indicator strips for steam sterilization are designed to monitor the attainment of the all critical variables for the specified sterilization processes. The STERIKING® AC6 strips are class 6 emulating indicators and intended to verify the steam sterilization cycle at 134°C for 3.5 minutes.

Conformity to International Standards

The STERIKING® AC Strips are manufactured under ISO 9001:2000 and ISO 13485:2003 certification and are in accordance with the following norms: ISO 11140-1:2005 (Class 6: emulating indicators).

STERIKING® monitoring products are designed, validated, and manufactured to suit their intended purposes.

Technical Data & Performance Characteristics

STERIKING® emulating indicator strips are manufactured from laminated printed carton. The AC6 strip is 10 cm long. After exposure to the specified processing conditions they provide a clear color change which remains unchanged and as such is suitable for use as a permanent record. The emulating indicator strips are printed with a reference color to show the endpoint reaction from green to brown.

Storage Recommendation & Shelf Life

No special precautions need to be taken in respect of storage or transportation of STERIKING® indicator strips. However, they should not be exposed to very high humidity or to direct sunlight. It is recommended to keep them in their original packing until the moment of use.

If stored according to the instructions the shelf life of STERIKING® AC strips is at least three years from the date of manufacture.

Sales and Transport Packing

STERIKING® indicator strips are packed in recyclable, brown hard carton cases (10 000 strips each), consisting smaller units of 250 strips each carton box.

Labelling: Each case bears a label with the necessary information/instructions of the content in accordance with ISO 11140-1:2005.

In Case of Complaint

In event of any complaint, the lot number and identification code must be provided by the complainant. For evaluation of claimed product, a defective sample (or a digital photo) and description of the defect together with an unused specimen must be made available to Wipak.

STERIKING® is a registered trademark of Wipak Medical.

Steriking® AC6 Strips

Code	Strips/case
AC6	250, 40x250

This specification refers to the named product group and shall be valid until the next revision. Other product related documents may be available upon request.

The information contained here is to our knowledge accurate and reliable as of the date of the publication. Wipak extends no warranties and makes no representations as to the accuracy or completeness of the information contained herein, and assumes no responsibility regarding the consequences of its use or for any printing errors. It is the customer's responsibility to inspect and test our products in order to satisfy himself as to the suitability of the products for the customer's particular purpose and suitability to the actual circumstances the product is exposed to. The customer is also responsible for the appropriate, safe and legal use, processing and handling of our products especially when recommendations for safe use and storage are given. Nothing herein shall constitute any warranty, nor is a protection from any law or patent to be inferred.